Ambu® PEEP 20 valve for single patient use

Corporate Head Office & Manufacturer:

Ambu A/S
Baltorpbakken 13
P.O. Box 232
DK-2750 Ballerup
Denmark
Tel.: +45 72 25 20 00
Fax: +45 72 25 20 50
E-mail: ambu@ambu.com
www.ambu.com

USA

Ambu Inc.
611 N. Hammonds Ferry Road
Linthicum, MD 21090-1356
Tel.: +1 410 636 1144
Fax: +1 410 636 9969
www.ambu.com

France

Ambu France S.A.R.L
Airspace – Rue Gagarine
F-33185 Le Haillan
Tel.: +33 5 57 92 31 50
Fax: +33 5 57 92 31 59
www.ambu.com

Germany

Ambu (Deutschland) GmbH
Strassheimer Strasse 1
D-61169 Friedberg
Tel.: +49 6031 7374-0
Fax: +49 6031 7374-20
www.ambu.com

UK

Ambu Ltd.
Burrell Road
St. Ives
Cambridgeshire PE27 3LE
Tel.: +44 1 480 498 403
Fax: +44 1 480 498 405
www.ambu.com

Italy

Ambu S.r.l
Via Toscana 8
I-20060 Vignate (MI)
Tel.: +39 0295 360 415
Fax: +39 0295 360 477
www.ambu.com

Sweden

AB Germa
Industrigratan 54-56
S-29 102 Kristiansand
Tel.: +46 4412 3030
Fax: +46 4410 3179
www.ambu.com

482 2038 00 - 02/2003 - Version 1.0.0 - Ambu A/S, Denmark


English: Directions for use. **Ambu® PEEP 20 valve for single patient use**
Deutsch: Bedienungsanleitung. **Ambu® PEEP Ventil 20 zum Einmalgebrauch**
Français: Mode d'emploi. **Valve Ambu® PEEP 20 à usage unique**
Español: Instrucciones de uso. **Válvula PEEP 20 Ambu® de un solo uso**
Svenska: Bruksanvisning. **Ambu® PEEP 20-ventil för endast en patient**
Italiano: Istruzioni per l'uso. **Valvola Ambu® PEEP 20 singolo paziente**
Português: Instruções de utilização. **Válvula Ambu® PEEP 20 para utilização única**
Dansk: Brugsanvisning. **Ambu® PEEP 20 ventil til énpatienters brug**
Nederlands: Gebruiksaanwijzing. **Ambu® PEEP 20 ventiel voor gebruik bij één**
Suomi: Käyttöohje. **Ambu® PEEP 20 -ventiili yhdelle potilaalle**
Norsk: Brukerveiledning. **Ambu® PEEP 20 ventil for en pasients bruk**
Ελληνικά: Οδηγίες Χρήσεως. **Ambu® PEEP 20 θαλάσσια μιας χρήσης**

□ 199 102 001

□ 199 103 001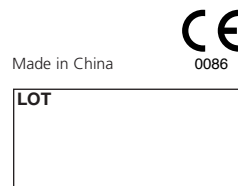

English: These directions for use may be updated without further notice. Copies of the current version are available from the manufacturer. **Ambu®** and **Ambu®** are registered trademarks of Ambu A/S, Denmark. Ambu A/S is certified according to ISO 9001, ISO 13485 and EN 46001.

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English

Ambu® PEEP 20 valve for single patient use

1. Introduction

The Ambu PEEP 20 valve is suitable for use with:

- Resuscitators
- Ventilators, if specified by the manufacturer
- CPAP systems, if specified by the manufacturer.
- When used on resuscitators the PEEP valve influences neither the inspiratory O₂ concentration nor the inspiratory resistance of the system. The PEEP valve can therefore remain at the patient valve during both spontaneous breathing and resuscitation.

2. Warnings/Precautions

- The PEEP valve should only be used by trained persons who are aware that PEEP may have an adverse effect on the patient's circulation.
- The PEEP 20 valve should only be used in situations where the patient's condition can be checked carefully by means of blood gas and haemodynamic tests.
- Do not use the PEEP valve on ventilators or breathing systems if the attachment of such valves is not specified by the manufacturer.
- PEEP levels must be monitored via a pressure manometer connected to the breathing system.
- For single patient use only. Reuse can cause cross-infection. Do not soak, rinse, or sterilize this device as these procedures may leave harmful residues. Destroy after use.

3. Specifications

The Ambu PEEP 20 valve is in conformity with Council Directive 93/42/EEC concerning Medical Devices

Connector size: ID 30 mm
Adapter size: OD 22 mm
Adjustment range: 0.15-2.0 kPa (1.5-20 cm H₂O)
Dimensions: Length 65 mm, diameter 45 mm
Weight: 40 g
Recommended operating temperatures: -20° to +50°C
Recommended storage temperatures: -40° to +70°C

4. Operating instruction

Inspect the PEEP 20 valve to make sure it is free of obstructions. After connecting the PEEP 20 valve to the equipment adjust the valve to the required PEEP level by turning the cap. For the adjustment of the valve, a manometer should be connected to the patient breathing for monitoring of the adjustment from 0.15-2.0 kPa (1.5-20 cm H₂O)

5. Trouble-shooting

| Problem | Possible cause | Correction |
|--|----------------------|--|
| It is not possible to obtain correct adjustment of the required PEEP level | Leak in the system | Check all the connections systematically |
| Valve noise | Insufficient damping | Replace the valve |

6. Complete PEEP valves

199 102 001 PEEP 20
199 103 001 PEEP 20, with 22 mm adaptor

Français

Valve Ambu® PEEP 20 à usage unique

1. Introduction

The Ambu PEEP 20 valve is suitable for use with:

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