Ambu® silikonimask

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| REF | Description |
|---------------------------------|---|
| <input type="checkbox"/> 251001 | Infant/Neonate face mask w/15 mm OD connector |
| <input type="checkbox"/> 251002 | Infant face mask w/15 mm OD connector |
| <input type="checkbox"/> 251003 | Child face mask w/22 mm ID connector |
| <input type="checkbox"/> 251004 | Large child face mask w/22 mm ID connector |
| <input type="checkbox"/> 251005 | Small adult face mask w/22 mm ID connector |
| <input type="checkbox"/> 251006 | Large adult face mask w/22 mm ID connector |

Lot Number:

Directions For Use

Ambu® Silicone Face Masks are for use with manual resuscitators, automatic resuscitators and ventilators.

| Materials: | LOT |
|-----------------|----------------------|
| Silicone Rubber | <input type="text"/> |

Lot Number:

Material:

| Cleaning and Sterilization - Reusable Item ONLY |
|---|
| To ensure safety of all kinds, please clean and sterilise |
| a. Between patients, |
| b. Whenever the product becomes contaminated, |
| c. Every 24 hours of use with the same patient. |

Carry out cleaning and sterilization as follows:

- Disassembly the product according to the instructions in the manual, if necessary.
- Wash the components thoroughly in warm water containing a mild detergent. Ensure that the detergent is compatible with the component materials.
- Rinse the components thoroughly in warm water to remove all traces of detergent or fluid.
- Sterilise the components using one of the following methods:
 - Autoclave up to 134 °C or 273 °F.
 - Ethylene Oxide.
 - Most common disinfecting solutions that are used for equipment coming into contact with the patient. Rinse thoroughly with water after sterilization.
- Dry all components thoroughly after sterilization.
- Inspect all components for wear and tear or damage, and replace them as necessary. Then package appropriately to protect the device from soil and contamination.

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Ambu® are registered trademarks of Ambu A/S