## Directions for use

# Ambu® Silicone Face Masks

## Silicone masks with inflated cuff

### 1. Warnings/precautions

#### Caution:


- U.S. federal law restricts this device to sale by or on the order of a physician (USA only).
- For use by CPR trained personnel only. The proper application of a face mask to obtain a tight seal should be trained in particular. Make sure that such personnel are made familiar with the contents of this manual.

### 2. Description

**Ambu® Transparent Face Masks** sizes 0 (with boring), 0, 2, 3/4, 5 and 6 with inflated cushion are designed for use with manual resuscitators, automatic resuscitators and ventilators.

#### **Sizes 0, 2, 3/4, 5 and 6 with a 22 mm ISO connector**

The mask domes are transparent. The soft cuff is inflatable and the cuff inflation may be adjusted.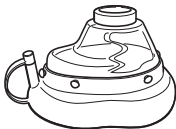

**Ambu** 

### **Size 0 with boring**

The model 0 with boring is designed for use with the **Ambu® Mark III resuscitator** on small children down to three years of age/approx. 15 kg of body weight. The leakage from the boring is designed to limit the ventilation pressure to approx. 4 kPa (40 cm H<sub>2</sub>O) **and shall not be used for any other application than that.**


## **3. Operating instructions**

### **Ventilation with a face mask**

Apply the mask firmly to the patient's face to achieve a tight seal. Hold the mask tight against the face while maintaining the correct head tilt to keep the airway open.

**It is very important to thoroughly train correct application of the face mask before any attempt is made to use a resuscitator.**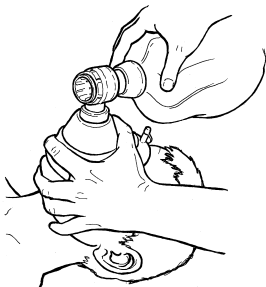

## 4. Cleaning - disinfecting - sterilising

● applicable ○ not applicable	Methods						
	Cleaning		Disinfecting - sterilising				
	Washing		Disinfecting			Auto-claving	
	Manual washing	Washing machine (WM)	W.M. heat disinfecting	Boiling	Chemical	121°C	138°C
<b>Mask</b> After each use	●	●	●	●	●	●	●

*The table shows applicable methods for cleaning, disinfecting and sterilising.*

### Detergents and chemical disinfectants

Numerous brands of detergents and chemical disinfectants are available. Use only brands that are compatible with the mask materials to avoid reduction in the lifetime of the materials. See list of materials on page 6. Follow the instructions of the manufacturer of the detergent or chemical disinfectant for dilution and exposure time. Substances containing phenol should be avoided.

Information on selected detergents and chemical disinfectants verified for compatibility with the mask materials is available on request from the local Ambu representative or Ambu A/S.

**Note:** If residuals of detergents or disinfectants are allowed to dry on the mask materials, surfaces may become impaired and the product lifetime may become reduced.


### The cleaning, disinfecting and sterilising process.

The following steps are generally recommended. Select proper methods for the mask parts in question according to the table above.


## 4.1. Disassembly of the mask (recommended)

Unbutton or remove mask cuff from dome.

**Caution!** Before autoclaving remove stopper from the inflation tube.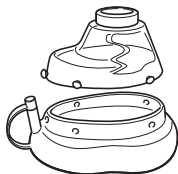

*Disassembled mask*

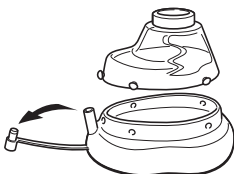*Disassembled mask with stopper removed from the inflation tube.*

## 4.2. Cleaning of parts

### Manual washing or automatic washing machine

Wash the parts in warm water using a detergent suitable for the mask materials cf. paragraph 5. Rinse all parts thoroughly in clean water to remove all detergent residues.

If surface cleaning and/or disinfecting of the outer surfaces of the mask are carried out, make sure that the detergent and/or disinfectant is compatible with the materials of the mask and be sure to remove the detergent/disinfectant by rinsing with water. If residues are allowed to dry on the mask, the lifetime of the materials may be reduced.

An automatic washing machine with a programme designed for washing anaesthesia breathing equipment may be used.

### **4.3. Disinfecting and/or sterilising**

Select heated or chemical disinfection according to local standards for disinfection and the table of applicable methods.

#### **Washing machine - Heat disinfecting programme**

Automatic washing machines designed for medical equipment will normally include program cycles for heat disinfection.

#### **Boiling**

Use clean water, heat and boil parts for e.g. 10 minutes to disinfect.

#### **Chemical disinfecting**

Follow the instructions of the manufacturer of the chemical disinfectant for dilution and exposure time. After exposure of the mask parts to the chemical disinfectant, rinse thoroughly in clean water to remove all residues.

#### **Autoclaving**

Use standard autoclaving equipment adjusted for max. 138 °C.

**Caution!** Before autoclaving remove stopper from the inflation tube.

### **4.4. Drying and cooling**

Leave the parts to dry and/or cool completely before reassembling the mask.

### **4.5. Inspection of parts**

After cleaning, disinfecting and/or sterilising carefully inspect all parts for damage or excessive wear and replace, if necessary. Some methods may cause discolouration of rubber parts without impact on their lifetime. In case of materials deterioration e.g. cracking, the parts should be replaced.

### **4.6. Assembling**

Reassemble the mask.

## 5. Specifications

### Weight

Size 0 with boring :	app. 39 g.	Size 3/4 :	app. 78 g.
Size 0 :	app. 38 g.	Size 5 :	app. 80 g.
Size 2 :	app. 63 g.	Size 6 :	app. 105 g.

Operating temperature range: -40°C to 50°C (-40°F to 122°F)

Storage temperature range: -40°C to 70°C (-40°F to 158°F)

Parts	Materials
Mask dome:	polysulphone
Mask cuff:	Silicone rubber
Mask connector:	Silicone rubber

Parts list		
Cat. No.	Size	Connector
Blue		
000 314 000	Face mask, size 0 with boring	22 mm ISO connector
000 315 000	Face mask, size 0	22 mm ISO connector
000 316 000	Face mask, size 2	22 mm ISO connector
000 312 000	Face mask, size 3/4	22 mm ISO connector
000 317 000	Face mask, size 5	22 mm ISO connector
000 319 000	Face mask, size 6	22 mm ISO connector

#### Note:

These directions for use may be updated without further notice.

Copies of the current version are available from the manufacturer.

**Manufacturer:**  
Ambu A/S  
Baltorbakken 13  
2750 Ballerup  
Denmark  
Tel.: +45 72 25 20 00  
Fax: +45 72 25 20 50  
E-mail: ambu@ambu.com  
www.ambu.com

**Subsidiary:**  
Ambu Inc.  
6740 Baymeadow Drive  
Glen Burnie, MD 21060  
Tel.: +1 410 768 6464  
Fax: +1 410 760 3993  
www.ambuUSA.com

**Ambu Ltd.**  
8 Burrell Road  
St. Ives  
Cambridgeshire PE27 3LE  
Tel.: +44 1480 498 403  
Fax: +44 1480 498 405  
www.ambu.com

#### Note:

These directions for use may be updated without further notice.

Copies of the current version are available from the manufacturer.

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